



New

Ref. F87319

Sinus Lift Kit

The Sinus Lift™ Kit offers the practitioner a straightforward and safe answer to the sinus lift operation. Thanks to the ultrasonic frequency modulation, there is no risk of damage to the membrane.

Used in conjunction with the Piezotome™, piezoelectric generator for implant surgery by Satelec®, these tips guarantee a fine, clean and effortless cut.

The diamond-coated Sinus Lift tips for window cutting and the three spatula tips for membrane detachment are cooled by the sterile spray (thus avoiding tissue damage from exposure to high temperatures). Moreover, the cavitation effect provides excellent visibility of the operating field.



SL1
Diamond scalpel (90µm)
 Cutting the buccal bone flap, smoothing sharp angles.
Ref. F87311



SL2
Ball diamond-coated tip (90µm)
 Cutting the buccal bone flap, precise osteoplasty. Ball diameter: 1.5mm
Ref. F87312



SL3
Flat-ended tip
 Detachment of the Schneider membrane from the window edge.
Ref. F87313



SL4
90° angled, flat-ended tip
 Detachment of the Schneider membrane inside the sinus.
Ref. F87314



SL5
120° angled, flat-ended tip
 Detachment of the Schneider membrane inside the sinus, removal of anatomical structures.
Ref. F87315



Recommended settings

	Piezotome Mode			
	1	2	3	4
SL1		X	X	
SL2		X	X	
SL3			X	X
SL4			X	X
SL5			X	X

Non contractual document - Ref. D87301-Q1 - Copyright © 2006 SATELEC. All rights reserved. No information or part of this document may be reproduced or transmitted in any form without the prior permission of SATELEC.

