


Suprasson®

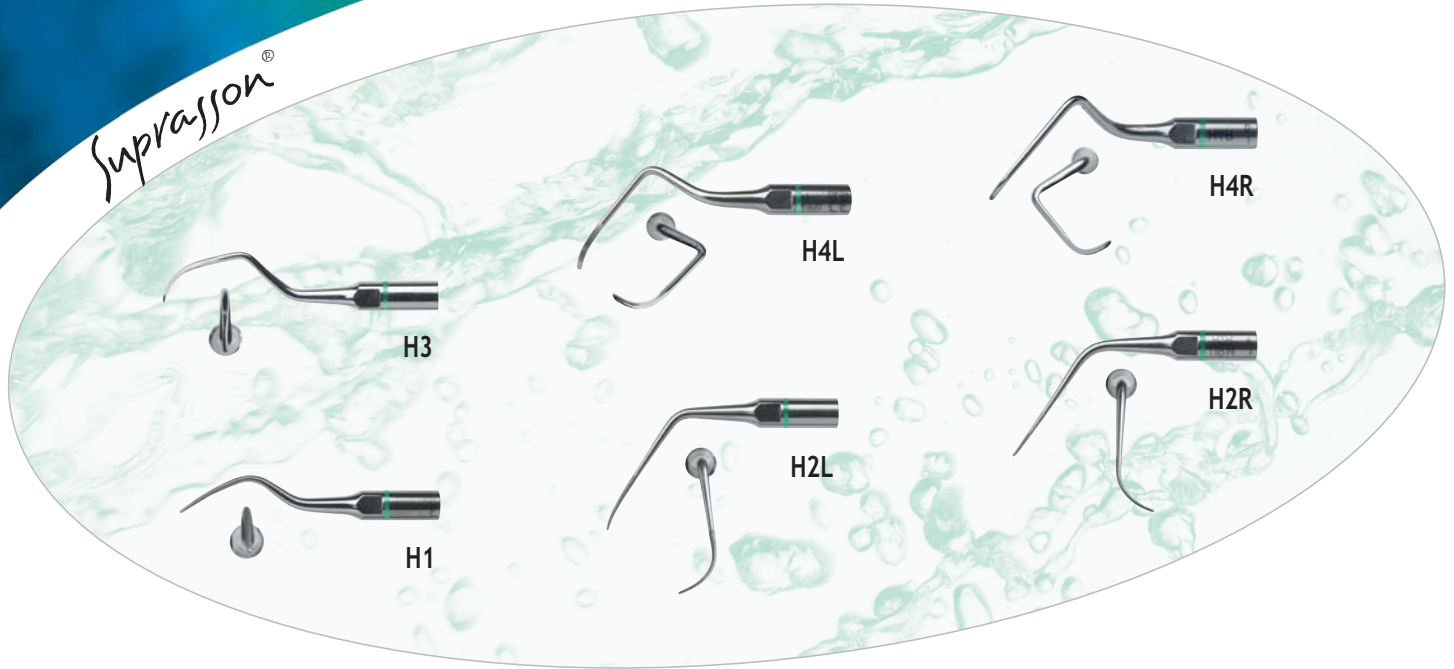


Perio debridement

Conseils d'utilisation - *Recommendations for use*

Débridement - *Debridement*





H3 : arc antérieur.
Anterior arch.

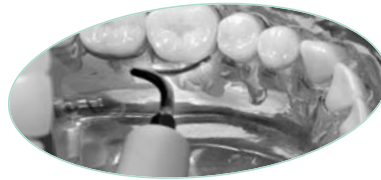


H4L/R : faces distales - furcations.
Distal faces - furcations.


- E : Email/Enamel
- D : Dentin(e)
- C : Cement(um)
- G : Gencive/Gingiva
- P : Poche/Pocket
- T : Tartre/Calculus
- H : Perio tip (H3-H4L/R)

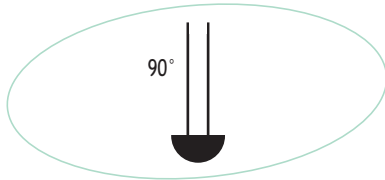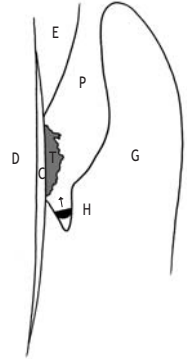

Mini-curette universelle.
Universal mini-curette.

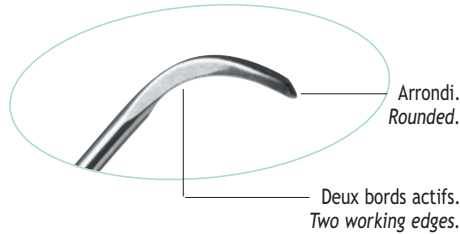


H4 : Gros plan (3/4).
Close-up (3/4).

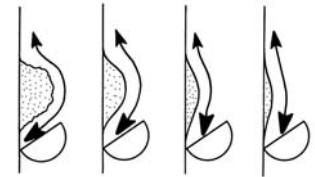

Guider en surfacage.
Guide the instrument («push-stroke»).

Guider en poussée - *Push stroke*


Surfaçage odontoplastie - *Surfacing odontoplasty*

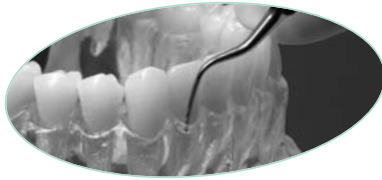

H1 : arc antérieur.
Anterior arch.



H2L/R : furcations.

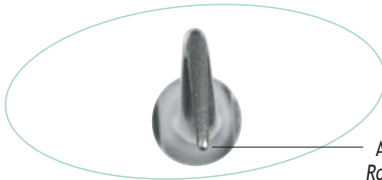


H1 : Gros plan.
Close-up.

Arrondi.
Rounded.



H2 : Gros plan (3/4).
Close-up (3/4).


Limes paro diamantées - *Perio diamond-coated files*

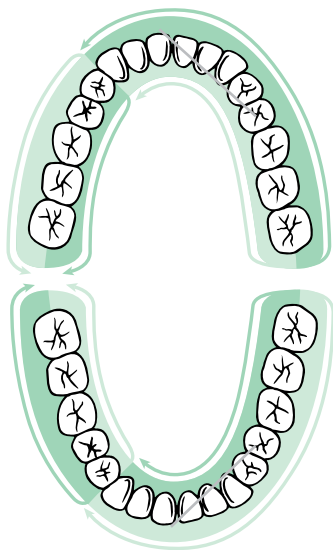





— Pivot 13 & 33 / Pivot at 13 & 33

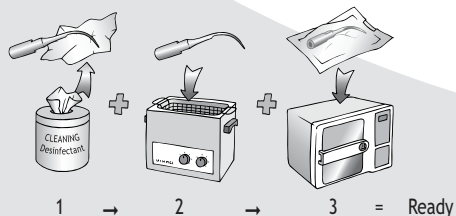■ H4L - H2L

■ H4R - H2R



				
	H1	H2L/R	H3	H4L/R
	Ref. F00366	L: Ref. F00367 R: Ref. F00368	Ref. F00369	L: Ref. F00114 R: Ref. F00115
P-Max Newtron XS	1-2	1-2	1-2	1-2
P5 Newtron / P5 Newtron XS	1-2	1-2	1-2	1-2
P-Max Newtron/Lux Prophy Max Newtron/Lux	1-5	1-5	1-5	1-5
P5 Booster	1-3	1-3	1-4	1-4
Module SP Newtron (LED)	1-2	1-2	1-2	1-2
Suprasson P-Max ^(P) /Lux ^(P) Prophy Max ^(P) /Lux ^(P)	1-5	1-5	5-10	5-10
Module SP4055 ^(P) /Lux ^(P)	1-5	1-5	1-5	1-5
Module SP3055 ^(E) /B ^(E)	1-3	1-3	1-3	1-3

(P): Mode Perio / Perio Switch.
(E): Mode Endo / Endo Switch



A Company of ACTEON Group • 17 av. Gustave Eiffel • BP 30216 • 33708 MERIGNAC cedex • FRANCE • Tel +33 (0) 556 34 06 07 • Fax +33 (0) 556 34 92 92

E-mail: satelec@acteongroup.com • www.acteongroup.com