# Bone Surgery

User's manual • Notice d'utilisation

PIEZOTOME™ - PIEZOTOME™ 2

IMPLANT CENTER - IMPLANT CENTER 2

# ENGLISH

# CONTENTS

**IMPORTANT** : Surgical tips designed for **PIEZOTOME** and **IMPLANT CENTER** cannot be used with **PIEZOTOME 2** and **IMPLANT CENTER 2** and vice versa.

I	Introduction	6
II	Description and clinical applications	7-8
III	Method of operation	8-9
IV	Sterilization	9
V	Replacing tips	10
	Subsidiaries	20-22

## I - INTRODUCTION

Specially designed to address periodontal and pre-implant surgery, the BONE SURGERY™ Kit allows bone cutting, exeresis and osteoplasty without any risk of damage to soft tissue.

Comprising six ultrasonic tips (three saws and three scalpels) adapted to the different cases that arise in the intra-oral environment, the BS Kit is used for clinical treatments such as bone harvesting, crest expansion, preparation of the implant site and accessing the lower alveolar nerve.

The sterile spray helps to cool the tips (thus avoiding tissue damage from exposure to high

temperatures) and provides excellent visibility of the operating field.

These tips guarantee a fine, clean and effortless cut.

**⚠ The PIEZOTOME tips and PIEZOTOME handpiece are designed for use with the PIEZOTOME generator or IMPLANT CENTER combined generator and bone surgery micro-motor, manufactured by Satelec.**

**The PIEZOTOME 2 tips and PIEZOTOME 2 handpiece are designed for use with the PIEZOTOME 2 generator or IMPLANT CENTER 2 combined generator and bone surgery micro-motor, manufactured by Satelec.**

## II - DESCRIPTION AND CLINICAL APPLICATIONS

### 2. 1 BS1 TIP

Bone saw with four teeth, used for cutting the cortical bone and bone distraction.

### 2. 2 BS2L TIP

Left-angled bone saw with four teeth, used for cutting the ramus cortical bone.

### 2. 3 BS2R TIP

Right-angled bone saw with four teeth, used for cutting the ramus cortical bone.

### 2. 4 BS4 TIP

Circular scalpel angled at 130°, for osteoplasty and harvesting of bone particles or chips. Treatments: crest osteoplasty, periodontal surgery, cyst exeresis...

## 2. 5 BS5 TIP

Flat scalpel for fine osteotomy.

Treatments: crest expansion, thin osteotomy, distraction, preparation of buccal bone flap on thick cortical terrain prior to sinus lift surgery.

## 2. 6 BS6 TIP

Curved scalpel particularly recommended for osteoplasty.

Treatments: remodelling, curettage and harvesting of bone chips.

## III - METHOD OF OPERATION

### 3. 1 FITTING THE TIPS

- Screw the selected tip to the PIEZOTOME handpiece with the dynamometric wrench\*;
- **Check that the spray is working properly.**

### 3. 2 SETTING THE MODE

#### a) PIEZOTOME - IMPLANT CENTER

1 being the most powerful mode

Tips	Recommended mode	Irrigation (ml/min)
BS1	1 2 3	40-50
BS2L	1 2 3	40-50
BS2R	1 2 3	40-50
BS4	1 2 3	35-40
BS5	1 2 3	35-40
BS6	1 2 3	35-40

Modes are set by the footswitch or on the front panel of the device.

#### b) PIEZOTOME 2 - IMPLANT CENTER 2

D1 being the most powerful mode

Tips	Recommended mode	Fine adjustment	Irrigation (ml/min)
BS1		01 02 03 04 ■■■■ 05	60
BS2L		01 02 03 04 ■■■■ 05	60
BS2R		01 02 03 04 ■■■■ 05	60
BS4		01 02 03 04 ■■■■ 05	60
BS5		01 02 03 04 ■■■■ 05	60
BS6		03 04 05 01 ■■■■ 02	60

Modes are set by the footswitch or on the front panel of the device.

## IV - STERILIZATION

### a) Lifetime

Tip shape and weight are the determining characteristics for obtaining maximal efficiency of the ultrasonic generator.

⚠ Careful monitoring of these two characteristics by the user will ensure that optimal performance of the device is maintained. Consequently, you are strongly advised not to alter the structure of the tips by filing or twisting them or modifying them in any other way.

⚠ Similarly, through normal wear, tip aging leads to alteration of its characteristics.

**Always replace a tip that has been damaged by wear or by accidental impact (dropped, bent, etc.).**

**These tips must be replaced when the cutting part becomes blunt or at least once a year.**

### b) Pre-disinfection / chemical cleaning

To be carried out immediately after treatment, wearing thick gloves.

Immerse in an ultrasonic bath (enzymatic, quaternary ammonium base detergent / disinfectant solution), carefully respecting the concentration and contact time recommended by the manufacturer of the solution.

Use a solution that bears the EC marking or that is in compliance with any standard required by the national regulations.

Rinse under running water for at least 30 seconds.

### **c) Mechanical / chemical cleaning**

To be carried out immediately after pre-disinfection/chemical cleaning, wearing thick gloves.

Scrub the products in a new bath of enzymatic, quaternary ammonium base detergent / disinfectant solution, for at least 30 seconds, and in any case until all remaining traces of contamination have been removed.

Rinse again under running water for at least 30 seconds.

### **d) Drying**

Dry with a clean, non-woven single-use cloth, so as to remove any traces of liquid.

Pack in single-use sterilization pouches or bags in compliance with the specifications defined in

standard EN ISO 11607-1 or in any equivalent standard required by the national regulations.

### **e) Sterilization**

The tips must be sterilized individually in an autoclave, according to the following parameters:

- autoclave: type B, compliant with standard EN 13060;
- sterilization temperature: 134°C;
- sterilization stabilization time: 18 minutes;
- pressure: 2 Bars min.

### **f) Storage**

Store the sterilized products in a dry place protected from dust.

Before re-use, if the integrity of the packaging is

not certain, repackage then re-sterilize in accordance with the defined protocol.

If contamination is visible inside the bag, destroy the product.

**g) Disposal of the product**

Dispose of the product in receptacles for waste materials of healthcare activities involving infectious risks.

**CAUTION: Not all autoclaves can reach 134°C. Not all autoclaves draw a pre-vacuum. Please refer to your autoclave manufacturer for specific sterilization instructions.**

**V - REPLACING TIPS**

**These tips must be replaced when the cutting part becomes blunt or at least once a year.**

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