Intralift

User's manual • Notice d'utilisation

PIEZOTOME™ - PIEZOTOME™ 2

IMPLANT CENTER - IMPLANT CENTER 2

ENGLISH

CONTENTS

IMPORTANT : Surgical tips designed for PIEZOTOME and IMPLANT CENTER cannot be used with PIEZOTOME 2 and IMPLANT CENTER 2 and vice versa.

I	Introduction	6
II	Description and clinical applications	7-8
III	Instructions for use	8-9
IV	Sterilization	9-11
V	Replacing tips	11
	Subsidiaries	20-22

I - INTRODUCTION

Developed for sinus lift by the crestal approach, the INTRALIFT™ kit makes it possible to realize non invasive surgery in full safety.

The diamond-coated tips, of increasing diameters (from 1.35mm to 2.80mm), are designed to drill and widen gradually the access canal to the Schneider membrane .

The sterile spray cools down the tips to avoid any rise in temperature, which could lead to tissue damage.

The membrane elevation is achieved using the TKW5 by means of microcavitation. Thanks to the ultrasonic frequency modulation, there is no risk of damage to the membrane.

Moreover, the cavitation effect enables excellent visibility of the operating field.

⚠ The PIEZOTOME tips and PIEZOTOME handpiece are designed for use with the PIEZOTOME generator or IMPLANT CENTER combined generator and bone surgery micro-motor, manufactured by Satelec.

The PIEZOTOME 2 tips and PIEZOTOME 2 handpiece are designed for use with the PIEZOTOME 2 generator or IMPLANT CENTER 2 combined generator and bone surgery micro-motor, manufactured by Satelec.

II - DESCRIPTION AND CLINICAL APPLICATIONS

2. 1 TKW1 TIP

Conical ($\varnothing 1.35\text{mm}$) and diamond-coated tip for pilot drilling. Drill the upper maxilla from the crest to the sinus floor.

2. 2 TKW2 TIP

Cylindrical ($\varnothing 2.1\text{mm}$) and diamond-coated tip dedicated to preliminary drilling and to widen the access canal to the Schneider membrane.

2. 3 TKW3 TIP

Cylindrical ($\varnothing 2.35\text{mm}$) and diamond-coated tip dedicated to preliminary drilling and to widen the access canal to the Schneider membrane.

2. 4 TKW4 TIP

Cylindrical ($\varnothing 2.80\text{mm}$) and diamond-coated tip dedicated to secondary drilling and to widen the access canal to the Schneider membrane.

2. 5 TKW5 TIP

Non cutting tip, that delivers sterile irrigation spray right up to the end, used for Schneider membrane elevation by means of microcavitation. The membrane elevation is

achieved gradually by a successive increase of irrigation flow rate. The TKW5 can also be used for compacting bone filling materials.

Never put the vibrating TKW5 tip in contact with the membrane.

III - INSTRUCTIONS FOR USE

3. 1 FITTING THE TIPS

- Screw the selected tip onto the PIEZOTOME handpiece with the universal wrench.
- **Check that the spray is working properly.**

3. 2 SETTING THE MODE

a) PIEZOTOME - IMPLANT CENTER

1 being the most powerful mode

Tips	Recommended mode	Irrigation (ml/min)
TKW1	1	80
TKW2	1	80
TKW3	1	80
TKW4	1	80
TKW5	2 3	40-50-60 ≤5 sec
	4 *	40 max. : 3-7 sec

* Mode 4 dedicated to compacting filling materials.
Modes are set by the footswitch or on the front panel of the device.

b) PIEZOTOME 2 - IMPLANT CENTER 2

D1 being the most powerful mode

Tips	Recommended mode	Fine adjustment	Irrigation (ml/min)
TKW1		02 <small>01 ■■■■■ 05</small>	80-100
TKW2		02 <small>01 ■■■■■ 05</small>	80-100
TKW3		02 <small>01 ■■■■■ 05</small>	80-100
TKW4		02 <small>01 ■■■■■ 05</small>	80-100
TKW5		01 02 03 <small>04 ■■■■■ 05</small>	40-50-60
	 *	01 02 03 <small>04 ■■■■■ 05</small>	40 max. : 3-7 sec

* Mode 4 dedicated to compacting filling materials.
Modes are set by the footswitch or on the front panel of the device.

IV - STERILIZATION

a) Lifetime

Tip shape and weight are the determining characteristics for obtaining maximal efficiency of the ultrasonic generator.

⚠ Careful monitoring of these two characteristics by the user will ensure that optimal performance of the device is maintained. Consequently, you are strongly advised not to alter the structure of the tips by filing or twisting them or modifying them in any other way.

⚠ Similarly, through normal wear, tip aging leads to alteration of its characteristics.

Always replace a tip that has been damaged by wear or by accidental impact (dropped, bent, etc.).

Replace routinely-used tips at least once to three times a year.

b) Pre-disinfection / chemical cleaning

To be carried out immediately after treatment, wearing thick gloves.

Immerse in an ultrasonic bath (enzymatic, quaternary ammonium base detergent / disinfectant solution), carefully respecting the concentration and contact time recommended by the manufacturer of the solution.

Use a solution that bears the EC marking or that is in compliance with any standard required by the national regulations.

Rinse under running water for at least 30 seconds.

c) Mechanical / chemical cleaning

To be carried out immediately after pre-disinfection / chemical cleaning, wearing thick gloves.

Scrub the products in a new bath of enzymatic, quaternary ammonium base detergent / disinfectant solution, for at least 30 seconds, and in any case until all remaining traces of contamination have been removed.

Rinse again under running water for at least 30 seconds.

d) Drying

Dry with a clean, non-woven single-use cloth, so as to remove any traces of liquid.

Pack in single-use sterilization pouches or bags in compliance with the specifications defined in

standard EN ISO 11607-1 or in any equivalent standard required by the national regulations.

e) Sterilization

The tips must be sterilized individually in an autoclave, according to the following parameters:

- autoclave: type B, compliant with standard EN 13060;
- sterilization temperature: 134°C;
- sterilization stabilization time: 18 minutes;
- pressure: 2 Bars min.

f) Storage

Store the sterilized products in a dry place protected from dust.

Before re-use, if the integrity of the packaging is

not certain, repackage then re-sterilize in accordance with the defined protocol.

If contamination is visible inside the bag, destroy the product.

g) Disposal of the product

Dispose of the product in receptacles for waste materials of healthcare activities involving infectious risks.

CAUTION: Not all autoclaves can reach 134°C. Not all autoclaves draw a pre-vacuum. Please refer to your autoclave manufacturer for specific sterilization instructions.

V - REPLACING TIPS

TKW1, TKW2, TKW3, TKW4 tips

These tips must be replaced when the diamond-coated cutting part becomes smooth and shiny or, at least, once a year.

TKW5 tip

This tip must be replaced when the cutting part becomes blunt or, at least, once a year.

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