# Sinus Lift

User's manual • Notice d'utilisation

PIEZOTOME™ - PIEZOTOME™ 2

IMPLANT CENTER - IMPLANT CENTER 2

# ENGLISH

# CONTENTS

**IMPORTANT : Surgical tips designed for PIEZOTOME and IMPLANT CENTER cannot be used with PIEZOTOME 2 and IMPLANT CENTER 2 and vice versa.**

I	Introduction	6
II	Description and clinical applications	7-8
III	Method of operation	8-9
V	Sterilization	9
VI	Replacing tips	10
	Subsidiaries	52-53

## I - INTRODUCTION

The SINUS LIFT Kit offers the practitioner a straightforward and safe answer to the sinus lift operation. Thanks to the ultrasonic frequency modulation, there is no risk of damage to the membrane.

The diamond-coated Sinus Lift tips for window cutting and the three spatula tips for membrane detachment are cooled by the sterile spray (thus avoiding tissue damage from exposure to high temperatures). The use of a manual instrument is recommended to control detachment of the membrane. Moreover, the cavitation effect provides excellent visibility of the operating field.

These tips guarantee a fine, clean and effortless cut.

**⚠ The PIEZOTOME tips and PIEZOTOME handpiece are designed for use with the PIEZOTOME generator or IMPLANT CENTER combined generator and bone surgery micro-motor, manufactured by Satelec.**

**The PIEZOTOME 2 tips and PIEZOTOME 2 handpiece are designed for use with the PIEZOTOME 2 generator or IMPLANT CENTER 2 combined generator and bone surgery micro-motor, manufactured by Satelec.**

## II - DESCRIPTION AND CLINICAL APPLICATIONS

### 2. 1 SL1 TIP

Diamond-coated tip for cutting the buccal bone flap and smoothing sharp angles.

### 2. 2 SL2 TIP

Diamond-coated tip for cutting the buccal bone flap (very thin bone) and precise osteoplasty.

Ball diameter: 1.5mm

Laser marking every 2 mm.

### 2. 3 SL3 TIP

This flat-ended non cutting tip is used for detaching the Schneider membrane from the window edge.

### 2. 4 SL4 TIP

Non cutting, right-angled tip, used for detaching the Schneider membrane inside the sinus.

## 2. 5 SL5 TIP

Non cutting tip angled at  $\pm 135^\circ$ , used for detaching the Schneider membrane inside the sinus and for removing anatomical structures.

## III - METHOD OF OPERATION

### 3. 1 FITTING THE TIPS

- Screw the selected tip to the PIEZOTOME handpiece with the dynamometric wrench\*;
- **Check that the spray is working properly.**

\*non delivered with the kit

## 3. 2 SETTING THE MODE

a) PIEZOTOME - IMPLANT CENTER

1 being the most powerful mode

Tips	Recommended mode	Irrigation (ml/min)
SL1	2 3	40-50
SL2	2 3	40-50
SL3	3 4	35-40
SL4	3 4	35-40
SL5	3 4	35-40

Modes are set by the footswitch or on the front panel of the device.

b) PIEZOTOME 2 - IMPLANT CENTER 2  
D1 being the most powerful mode

Tips	Recommended mode	Fine adjustment	Irrigation (ml/min)
SL1		01 02 03 01 ■■■■■ 05	60
	 	01 - 05 01 ■■■■■ 05	
SL2		01 02 03 01 ■■■■■ 05	60
	 	01 - 05 01 ■■■■■ 05	
SL3		01 02 03 01 ■■■■■ 05	50
SL4		01 02 03 01 ■■■■■ 05	30
SL5		01 02 03 01 ■■■■■ 05	30

Modes are set by the footswitch or on the front panel of the device.

## IV - STERILIZATION

### a) Lifetime

Tip shape and weight are the determining characteristics for obtaining maximal efficiency of the ultrasonic generator.

⚠ Careful monitoring of these two characteristics by the user will ensure that optimal performance of the device is maintained. Consequently, you are strongly advised not to alter the structure of the tips by filing or twisting them or modifying them in any other way.

⚠ Similarly, through normal wear, tip aging leads to alteration of its characteristics.

Always replace a tip that has been damaged by wear or by accidental impact (dropped, bent, etc.).

Replace routinely-used tips at least once a year.

**b) Pre-disinfection / chemical cleaning**

To be carried out immediately after treatment, wearing thick gloves.

Immerse in an ultrasonic bath (enzymatic, quaternary ammonium base detergent / disinfectant solution), carefully respecting the concentration and contact time recommended by the manufacturer of the solution.

Use a solution that bears the EC marking or that is in compliance with any standard required by the national regulations.

Rinse under running water for at least 30 seconds.

**c) Mechanical / chemical cleaning**

To be carried out immediately after pre-disinfection/chemical cleaning, wearing thick gloves.

Scrub the products in a new bath of enzymatic, quaternary ammonium base detergent / disinfectant solution, for at least 30 seconds, and in any case until all remaining traces of contamination have been removed.

Rinse again under running water for at least 30 seconds.

**d) Drying**

Dry with a clean, non-woven single-use cloth, so as to remove any traces of liquid.

Pack in single-use sterilization pouches or bags in compliance with the specifications defined in

standard EN ISO 11607-1 or in any equivalent standard required by the national regulations.

#### **e) Sterilization**

The tips must be sterilized individually in an autoclave, according to the following parameters:

- autoclave: type B, compliant with standard EN 13060;
- sterilization temperature: 134°C;
- sterilization stabilization time: 18 minutes;
- pressure: 2 Bars min.

#### **f) Storage**

Then, store the sterilized products in a dry place protected from dust.

Before re-use, if the integrity of the packaging is not certain, repackage then re-sterilize in accordance with the defined protocol.

If contamination is visible inside the bag, destroy the product.

#### **g) Disposal of the product**

Dispose of the product in receptacles for waste materials of healthcare activities involving infectious risks.

**CAUTION: Not all autoclaves can reach 134°C. Not all autoclaves draw a pre-vacuum. Please refer to your autoclave manufacturer for specific sterilization instructions.**

## V - REPLACING TIPS

### SL1 and SL2 tips

These tips must be replaced when the diamond-coated cutting part becomes smooth and shiny or at least once a year.

### SL3, SL4 and SL5 tips

These tips must be replaced when the cutting part becomes blunt or at least once a year.

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