

Summary *PIEZOTOME* Piezo•Ultrasonic•Surgery•Unit

Tips	Maximum recommended modes	Irrigation ml/min
BS1	1 2 3 4	40-50
BS2L	1 2 3 4	40-50
BS2R	1 2 3 4	40-50
BS4	1 2 3 4	35-40
BS5	1 2 3 4	35-40
BS6	1 2 3 4	35-40
SL1	1 2 3 4	40-50
SL2	1 2 3 4	40-50
SL3	1 2 3 4	35-40
SL4	1 2 3 4	35-40
SL5	1 2 3 4	35-40

Summary

Piezo•Ultrasonic•Surgery•Unit

Tips	Maximum recommended modes	Irrigation ml/min
TKW1	1 2 3 4	80
TKW2	1 2 3 4	80
TKW3	1 2 3 4	80
TKW4	1 2 3 4	80
TKW5	1 2 3 4	30-40
LC1	1 2 3 4	60-80
LC1 90°	1 2 3 4	60-80
LC2	1 2 3 4	60-80
LC2L	1 2 3 4	60-80
LC2R	1 2 3 4	60-80
NINJA™	1 2 3 4	60-80
CE1	1 2 3 4	60-80
CE2	1 2 3 4	60-80
CE3	1 2 3 4	60-80