
































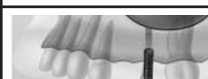

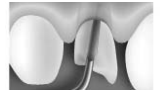



















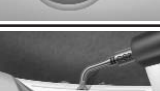

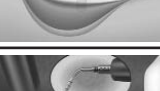

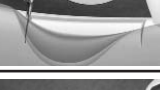






Piezotome 2 - Piezotome Solo - Implant Center 2

Name	Photo	Function	Use	Mode *	Irrigation
BS TIPS: BONE SURGERY					
BS1		Bone surgery		D1 3 MAX <small>01 02 03 04 05</small>	60ml
BS1 Long		In-depth bone grafting		D1 3 MAX <small>01 02 03 04 05</small>	60ml
BS2L		Ramus bone grafting		D1 3 MAX <small>01 02 03 04 05</small>	60ml
BS2R		Ramus bone grafting		D1 3 MAX <small>01 02 03 04 05</small>	60ml
BS4		Osteoplasty - Bone remodeling		D1 3 MAX <small>01 02 03 04 05</small>	60ml
BS5		Crest splitting		D3 3 MAX <small>01 02 03 04 05</small>	60ml
BS6		Osteoplasty - Bone remodeling		D1 3 MAX <small>01 02 03 04 05</small>	60ml
SL TIPS: SINUS LIFT BY THE LATERAL APPROACH					
SL1		Bone window cut		D1 3 MAX <small>01 02 03 04 05</small>	60ml
SL2		Bone window cut		D1 3 MAX <small>01 02 03 04 05</small>	60ml
SL3		Membrane detachment		D4 3 MAX <small>01 02 03 04 05</small>	50ml
SL4		Membrane detachment		D4 3 MAX <small>01 02 03 04 05</small>	30ml
SL5		Membrane detachment		D4 3 MAX <small>01 02 03 04 05</small>	30ml
TKW TIPS: SINUS LIFT BY THE CRESTAL APPROACH					
TKW1		Pilot drilling - Ø1.35mm		D2 3 MAX <small>01 02 03 04 05</small>	100ml
TKW2		Bone drilling - Sinus floor opening Ø2.10mm		D2 3 MAX <small>01 02 03 04 05</small>	100ml
TKW3		Bone drilling - Canal widening - Ø2.35mm		D2 3 MAX <small>01 02 03 04 05</small>	100ml
TKW4		Bone drilling - Receptacle - Ø2.80mm		D2 3 MAX <small>01 02 03 04 05</small>	100ml
TKW5		Membrane detachment		D3 1 MAX <small>01 02 03 04 05</small>	30-40ml
		Plug & Spray		D3 3 MAX <small>01 02 03 04 05</small>	10ml

* Fine setting (ex :3 MAX) only available on Piezotome 2 and Implant Center 2

Piezotome 2 - Piezotome Solo - Implant Center 2

Name	Photo	Function	Use	Mode *	Irrigation
LC TIPS: EXTRACTION					
LC1		Extraction		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
LC190°		Extraction		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
LC2		Extraction		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
LC2L		Extraction		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
LC2R		Extraction		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
Ninja		Extraction Amputation Hemisection		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
CE TIPS: CROWN EXTENSION					
CE1		Osteoplasty Ø1.75mm		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
CE2		Osteoplasty Ø1.20mm		D2 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
CE3		Osteoectomy Laser mark: 3mm		D1 3 MAX <small>D1 ■■■■■ 05</small>	60-80ml
CS TIPS: CREST EXPANSION					
CS1		Pilot crest splitting (8mm depth)		D3 Maxillary D2 Mandible 3 MAX <small>D1 ■■■■■ 05</small>	80-100ml
CS2		Crest splitting (8mm depth)		D3 Maxillary D2 Mandible 3 MAX <small>D1 ■■■■■ 05</small>	80-100ml
CS3		Discharge incisions (8mm depth)		D3 Maxillary D2 Mandible 3 MAX <small>D1 ■■■■■ 05</small>	80-100ml
CS4		Expansion Thickness: 1.80mm at 8mm		D3 Maxillary D2 Mandible 3 MAX <small>D1 ■■■■■ 05</small>	80-100ml
CS5		Expansion Thickness: 2.75mm at 8mm		D3 Maxillary D2 Mandible 3 MAX <small>D1 ■■■■■ 05</small>	80-100ml
CS6		Expansion Thickness: 3.75mm at 8mm		D3 Maxillary D2 Mandible 3 MAX <small>D1 ■■■■■ 05</small>	80-100ml

* Fine setting (ex :3 MAX) only available on Piezotome 2 and Implant Center 2