

# Summary

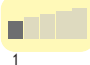
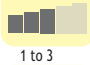
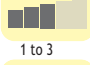


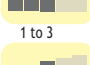



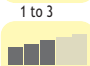
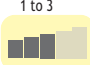
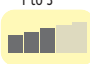


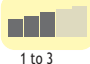



Tips	Maximum recommended modes	Fine setting*	Irrigation ml/min
BS1S	<b>D1</b> D2 D3 D4	1 to 3	60
BS1L	<b>D1</b> D2 D3 D4	1 to 3	60
BS2L	<b>D1</b> D2 D3 D4	1 to 3	60
BS2R	<b>D1</b> D2 D3 D4	1 to 3	60
BS4	<b>D1</b> D2 D3 D4	1 to 3	60
BS5	D1 D2 <b>D3</b> D4	1 to 3	60
BS6	<b>D1</b> D2 D3 D4	1 to 3	60
SL1	<b>D1</b> D2 D3 D4	1 to 3	60
SL2	<b>D1</b> D2 D3 D4	1 to 3	60
SL3	D1 D2 D3 <b>D4</b>	1 to 3	50
SL4	D1 D2 D3 <b>D4</b>	1 to 3	30
SL5	D1 D2 D3 <b>D4</b>	1 to 3	30
TKW1	D1 <b>D2</b> D3 D4	3	100
TKW2	D1 <b>D2</b> D3 D4	3	100
TKW3	D1 <b>D2</b> D3 D4	3	100
TKW4	D1 <b>D2</b> D3 D4	3	100

\*Not applicable to PIEZOTOME SOLO (LED).

# Summary



Tips	Maximum recommended modes	Fine setting*	Irrigation ml/min
TKW5	D1 D2 <b>D3</b> D4	 1	30-40
CS1	D1 <b>D2</b> <b>D3</b> D4 <small>Mandible Maxillary</small>	 1 to 3	80-100
CS2	D1 <b>D2</b> <b>D3</b> D4 <small>Mandible Maxillary</small>	 1 to 3	80-100
CS3	D1 <b>D2</b> <b>D3</b> D4 <small>Mandible Maxillary</small>	 1 to 3	80-100
CS4	D1 <b>D2</b> <b>D3</b> D4 <small>Mandible Maxillary</small>	 1 to 3	80-100
CS5	D1 <b>D2</b> <b>D3</b> D4 <small>Mandible Maxillary</small>	 1 to 3	80-100
CS6	D1 <b>D2</b> <b>D3</b> D4 <small>Mandible Maxillary</small>	 1 to 3	80-100
LC1	<b>D1</b> D2 D3 D4	 1 to 3	60-80
LC190°	<b>D1</b> D2 D3 D4	 1 to 3	60-80
LC2	<b>D1</b> D2 D3 D4	 1 to 3	60-80
LC2L	<b>D1</b> D2 D3 D4	 1 to 3	60-80
LC2R	<b>D1</b> D2 D3 D4	 1 to 3	60-80
NINJA™	<b>D1</b> D2 D3 D4	 1 to 3	60-80
CE1	<b>D1</b> D2 D3 D4	 1 to 3	60-80
CE2	D1 <b>D2</b> D3 D4	 1 to 3	60-80
CE3	<b>D1</b> D2 D3 D4	 1 to 3	60-80

\*Not applicable to PIEZOTOME SOLO (LED).